

That healing feeling

The geko® device doubles the rate of healing in venous leg ulcers versus compression alone¹

Reduces pain fast¹³
Accelerates healing¹
Improves concordance¹⁵

Now available
on prescription



Help patients feel like themselves again


geko®
wound therapy

The future of venous leg ulcer care can begin today

Approximately 1.5% of the adult population in the UK suffer from leg ulcers.² Many experience pain, anxiety, altered body image and isolation as a result of chronic, non-healing venous leg ulcers (VLUs).^{3,4}



Compression therapy is the gold standard of VLU treatment, but compression alone may not be appropriate or effective for every patient.^{6,9,10} Many patients find compression too painful to tolerate and adhere to. This is especially true of patients suffering with oedema which can often occur alongside VLUs.^{9,11}

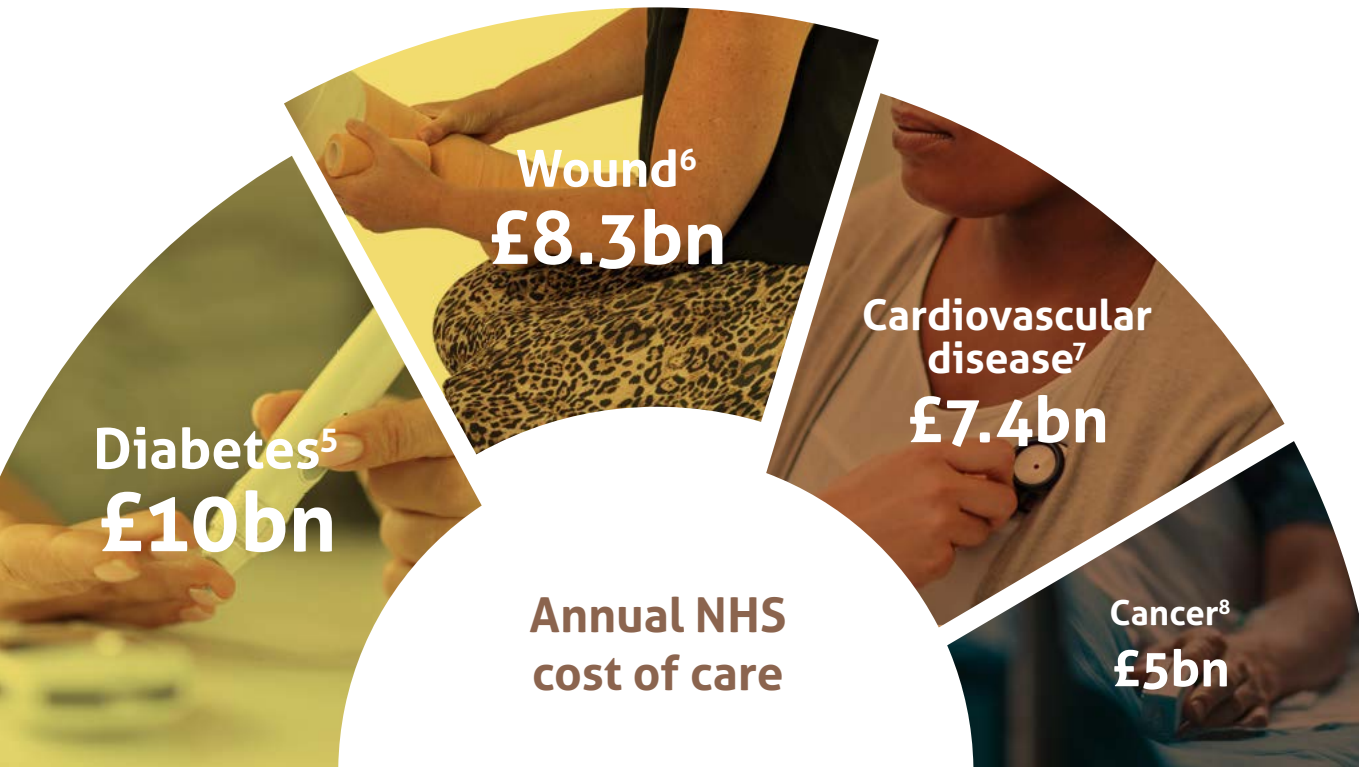

What can be done to minimise the future impact of VLU care on patients and your healthcare team?

A small device with big results clinically proven to significantly increase VLU healing rates¹

Introducing the geko[®] device:



Doubles the rate of healing in VLUs versus compression alone¹



With only 4 weeks of treatment with the geko device, 42% of wounds healed over the duration of the study¹



Reduced oedema¹² and 52% of patients reported a reduction in pain¹³



High patient adherence rate of 94%¹



The geko device boosts patients' hope and optimism towards their wound healing⁴

Technology that mimics the effect of walking

How does the geko[®] device work?

- The device is battery operated with no wires and is worn just below the knee for 12 hours per day
- It stimulates the common peroneal nerve via non-invasive skin electrodes^{9,14}
- The device delivers a low frequency pulse once per second^{9,11,14}
- This gentle stimulation causes the muscles in the calf and foot to contract, increasing venous and arterial flow^{11,13,15}
- Microcirculatory flow in the wound bed and wound periphery is substantially increased^{12,16}
- Improved blood circulation and oxygen delivery to the wound bed results in a **significantly faster rate of healing**^{1,11,17}



Faster healing that your patients can *feel*

Better patient engagement with the geko® device:



In one study, nurses found that their patients were more engaged in the care of their VLU's because they could *feel* the device working¹⁷



The device is wearable and easy to self-operate, giving patients and/or their carers more control over managing their treatment¹³



Ten device settings allow patients to adjust the intensity of their treatment, ensuring a visible foot twitch/movement whilst also meeting their comfort needs^{13,15}



You could feel when it was in the position it needed to be in...if it's in the right spot it's like a warm wave down the leg

Patient¹⁸



It's definitely easy to use...originally the nurse would mark out where it should go but now I have a better understanding of whereabouts the nerve is... I feel quite expert!

Caregiver¹⁸

Supporting you and your healthcare team

Managing unhealed wounds adds huge pressure on the NHS⁶



Over
£350
million

spent on wound care products annually
e.g. dressings, bandages and creams




97% more community
nurse visits

Healed in 16 weeks with
4 weeks of geko device use
Total treatment cost = **£2,892**



Using the
geko[®] device with
compression therapy
provides a **cost
saving of 63%** vs
compression alone^{*6,19}

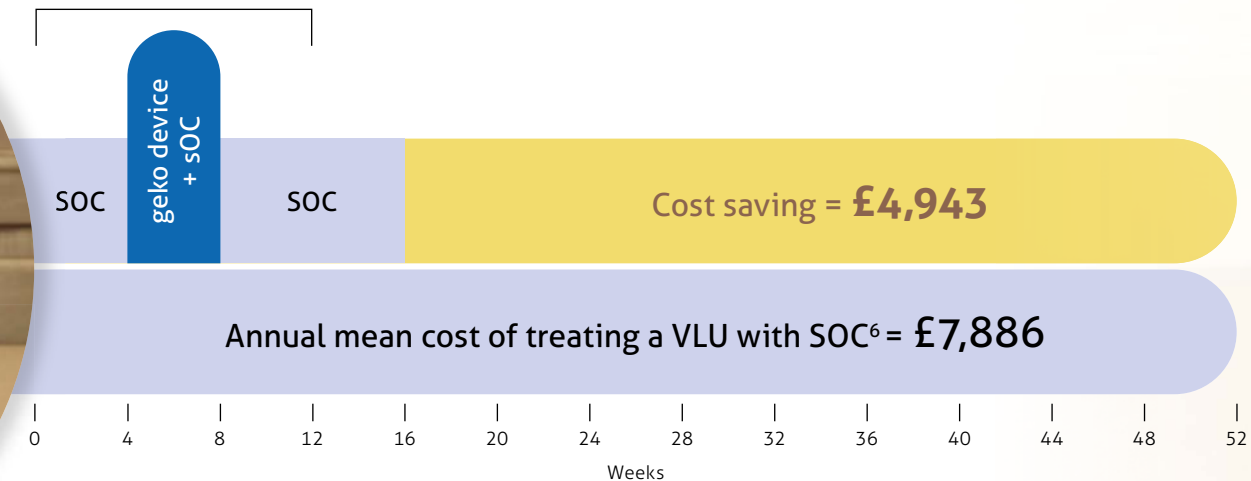

*Illustration of the cost of healing a VLU after 16 weeks following 4 weeks of geko treatment compared to the annual cost of treating a non-healing VLU with standard of care alone.

Who are the right patients for the geko[®] device?

The geko device works alongside compression therapy and can be used on patients when:



They have a VLU that is not on a positive healing trajectory or not responding to standard of care



They have a VLU with a static rate of healing with no sign of revascularisation of the wound bed



They are unable to tolerate compression therapy

**The geko device
doubles the rate of healing
in venous leg ulcers versus
compression alone¹**



Find out more about how the geko[®] device can support your patients and your healthcare team at www.gekocodevices.com

1. Bull R *et al.* Int Wound J. 2023; 1–9.
2. Guest J *et al.* BMJ Open 2015; 5: DOI: 10.1136/bmjopen-2015-009283.
3. Wounds International. Principles of compression in venous disease: A practitioner's guide to treatment and prevention of venous leg ulcers. Available at: <https://www.woundsinternational.com>. Accessed: April 2023.
4. Kuhnke JL and Maxwell S. Spotlight on Research 2021. Available at: <https://static1.squarespace.com/static/5cbe6fe5809d8e6a6dd160f0/t/60e738a9ef39b265666ba0e0/1625766061102/Kuhnke+%26+Maxwell%2C+%282021%29.pdf>. Accessed: April 2023.
5. NHS England. NHS prevention programme cuts chances of type 2 diabetes for thousands. Available at: <https://www.england.nhs.uk/2022/03/nhs-prevention-programme-cuts-chances-of-type-2-diabetes-for-thousands/>. Accessed: April 2023.
6. Guest J *et al.* BMJ Open 2020; 10: DOI: 10.1136/bmjopen-2020-045253.
7. Public Health England. Health matters: preventing cardiovascular disease. Available at: <https://www.gov.uk/government/publications/health-matters-preventing-cardiovascular-disease/health-matters-preventing-cardiovascular-disease>. Accessed: April 2023.
8. Department of Health & Social Care. 2010 to 2015 government policy: cancer research and treatment. Available at: <https://www.gov.uk/government/publications/2010-to-2015-government-policy-cancer-research-and-treatment/2010-to-2015-government-policy-cancer-research-and-treatment>. Accessed: April 2023.
9. Ingves M *et al.* J Investig Med 2014; DOI: 10.1177/2324709614559839.
10. Harris C *et al.* Int Wound J 2017; 14: 1100–07.
11. Das S *et al.* Int Wound J 2021; 18(2): 187–93.
12. Williams K *et al.* Vasc Health Risk Manag 2021; 17: 771–78.
13. Jones N *et al.* Br J Nurs 2018; 27(20): S16–S21.
14. geko[™] devices. OnPulse Technology. Available at: <https://www.gekocodevices.com/onpulse-technology/>. Accessed: April 2023.
15. Harris C *et al.* Int Wound J 2017; 14: 1189–98.
16. Das S *et al.* J Wound Care 2021; 30(2): 151–5.
17. Harris C *et al.* Int Wound J 2019; 16: 266–74.
18. geko[™] device data on file. Clinical Programme Feedback. Firstkind Limited. 2022.
19. geko[™] device data on file. Theoretical annual treatment cost calculation. Firstkind Limited. 2023.

Please refer to the Instructions For Use (IFU) for further details on fitting and warnings.



www.lightlymedical.com.au
email: info@lightlymedical.com.au
Tel: 1300 820 125
Address:
5/21 Eugene Terrace,
Ringwood 3134,
Victoria,
Australia

MDBRWTVNAU0795

geko[®]
wound therapy

